









ADIANT 2-PLY FACE COVERING USAGE AND CARE

GENERAL REQUIREMENTS





-  Employees who feel sick with symptoms of fever, cough, or shortness of breath must stay home, seek medical care, and avoid contact with others.
-  Employees developing respiratory symptoms, such as a cough during working hours must notify their supervisor/ HR and leave the workplace as soon as possible.
-  Employees shall still follow all other guidance provided by the company and official sources about COVID-19.

PROPER USE OF FACE COVER



-  **BEFORE** touching the face cover, clean hands with alcohol-based hand rub or soap and water.
-  **INSPECT** face cover for tears and holes.
-  **VERIFY** which side is the top (it is where the metal or stiff edge piece is).
-  **IDENTIFY** the outside of face cover that has the blue BioSmart® fabric (see picture to the right).
-  **AVOID** touching the face cover while using it; if you do, clean your hands with alcohol-based hand rub or soap and water.



HOW TO PUT ON A FACE COVER

-  **FIT** face cover over your nose, mouth and chin with the metal nose slip on top and the blue BioSmart® fabric facing outwards.
-  **TIE** the straps securely behind your head and neck
-  **PINCH** the metal strip or stiff edge so it holds to the shape of your nose.
-  **ADJUST** the face cover over your mouth, nose, and chin and make sure there are no gaps between your face and the cover.

REMOVE THE FACE COVER

-  **REMOVE** your face cover by loosening the ties from the back of your head and pull the ties forward — do not touch the front of the face cover. If you do, please remember to wash your hands with soap and water before and after you touch your face cover.
-  **TREAT** the front of the face cover as contaminated. **CLEAN** your hands with alcohol-based hand rub or soap and water after removal.

DISINFECTING YOUR 2-PLY FACE COVER



OPTION 1:

1. Launder in a washing machine with EPA registered bleach such as the following Clorox products: Germicidal Bleach₄, Regular Bleach₂ with CLOROMAX®, Performance Bleach₂ with CLOROMAX®, Disinfecting Bleach with CLOROMAX®. Launder in a washing machine according to the 'laundry cleaning & sanitizing' instructions on the bleach container.
2. **ALLOW** to dry thoroughly before use or storage. Dry by air or in clothes dryer on high/cotton temperature setting.
3. **STORE** dried face cover in a clean, sealed plastic bag when not in use.
4. **ENSURE** hands are clean when handling disinfected face cover.



OPTION 2:

1. **PREPARE** a disinfecting solution by mixing sodium hypochlorite (5.25% bleach) and water:
 - 8 tablespoons (1/2 cup) bleach per gallon of water
 - Or
 - 2 tablespoons bleach per quart of water.

NOTE: DO NOT use the bleach full strength and be careful not to splash bleach/water solution as this can damage clothing. Bleach degrades with age and exposure to heat and sunlight, so use fresh bleach and store away from heat and sunlight.

2. **USE RUBBER GLOVES** and avoid contact of disinfecting solution with skin and other items.
3. **SOAK** face cover in disinfecting solution for minimum of **10** minutes, rinse thoroughly in water.
4. **ALLOW** to dry thoroughly before use or storage. Dry by air or in clothes dryer on high/cotton temperature setting.
5. **STORE** dried face cover in a clean, sealed plastic bag when not in use.
6. **ENSURE** hands are clean when handling disinfected face cover.



OPTION 3*:

1. **HEAT** dry for at least 30 minutes.
2. **USE** hot/cotton temperature setting in clothes dryer.
3. **STORE** dried face cover in a clean, sealed plastic bag when not in use.

***If Options 1 and 2 are not available, then follow Option 3 to kill viral contamination.**

However, washing with a bleach/water solution is always preferred. This is to maintain anti-microbial properties.